F

INDICATIONS

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UK

INDICATIONS

ADHERENCE® CC is indicated for the following applications:

- Final cementation of implant prostheses, crowns and bridges, inlays and onlays, Maryland bridges, veneers.
- Final cementation for pins, posts.
- Periodontal splinting.

PROPERTIES

- ADHERENCE® CC provides high compressive and flexural strength and low solubility required for these types of restorations.
- ADHERENCE® CC is designed to meet the requirements per the ISO 4049:2000 standard (Type 2, Class 1 product).

FORMULATION

- Catalyst: BisGMA based resin system with glass filler (70wt%) and peroxide initiator
- Base: BisGMA based resin system with glass filler (65wt%) and amine catalyst

DIRECTIONS FOR USE

When ready to use:

- Dispense equal volumes of catalyst and base.
- Close syringes with appropriate cap immediately after use.
- Mix together equals parts of the **catalyst** and **base** for 30 seconds.
- After placement of the restoration, hold it in place for 3-4 minutes (initial set), but can start cement clean up at 1.5 to 2.5 minutes when material becomes rubbery.
- Adjust or allow to function (final set) after 4 to 6 minutes.

I - Ceramic or composite indirect restoration to enamel/ dentin

- After try-in, place etched ceramic restoration in acetone or ethanol in ultrasonic cleaner for 1-2 minutes.
- Place silane-coupling agent on internal surface of porcelain restoration.
- Clean tooth with non-fluoridated prophy paste or flour of pumice.
- Use a dual-curing bonding agent. Avoid the use of self etch-primer adhesive.** Etch and apply the bonding system according to instructions.
- Prepare the ADHERENCE® CC as indicated above and place in restoration using plastic instrument or injection system. Be sure not to trap any air bubbles.
- Seat restoration in place on tooth, clean up excess cement, and hold in place until ADHERENCE® CC reaches initial set.
- Adjust or allow to function after final set.

II - Metal to enamel/dentin

Metal restorations to teeth-crowns, inlays, bridges, and posts including Maryland bridges.

- After try-in, sandblast tooth side of restoration, wash and dry.
- If desired, use a metal priming system of your choice to treat the metal exposed to the ADHERENCE® CC.
- Clean tooth with non-fluoridated prophy paste or flour of pumice.
- Use a dual-curing bonding agent. Avoid the use of self etch-primer adhesive.** Etch and apply the bonding system according to instructions.
- Prepare the ADHERENCE® CC as indicated above and place in restoration using plastic instrument or injection system. Be sure not to trap any air bubbles.
- Seat restoration in place on tooth, clean up excess cement, and hold in place until ADHERENCE® CC reaches initial set.
- Adjust or allow to function after final set.

III - Metal to Metal

Metal to metal cementation (implant prosthesis or crown on post)

- Use an air abrasive system (sandblasting) on all external metal surfaces to be exposed to ADHERENCE® CC. Intraoral metal should be roughened with a bur or micro-etcher.
- If desired, apply a metal priming system of your choice to prepare the external metal for the ADHERENCE® CC.
- Clean posts or abutments with non-fluoridated flour of pumice prior to cementation to remove any plaque or other debris from the surface.
- Prepare the ADHERENCE® CC as indicated above and place in restoration using plastic instrument or injection system. Be sure not to trap any air bubbles.
- Seat restoration in place on tooth, clean up excess cement, and hold in place until ADHERENCE® CC reaches initial set.
- Adjust or allow to function after final set.

WARNINGS

- Contains methacrylate resins. Avoid use of this product on patients with known methacrylate allergies. To reduce the risk of allergic response, avoid exposure to uncured resin.
- Do not use eugenol-containing materials for pulp protection and temporary sealing since they can retard the curing process.
- Any non-sterilizable item that is handled in the delivery of the dental service should be disinfected by standard dental office hygiene procedures.

- Xi- Irritant

 - May cause sensitisation by skin contact.
 - Irritating to eyes and skin.
 - Avoid contact with skin.
 - Wear suitable gloves.
 - Wear eye/face protection.
 - In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

STORAGE

- 2°C – 8°C
- Allow cement to come to room temperature before use.
- Do not store in proximity to eugenol-containing products.
- Do not expose the cement to elevated temperatures.

PRESENTATION

Kit:

- 1 syringe x 4.5 g base
- 1 syringe x 4.5 g catalysor
- 1 Mixing pad
- plastic spatulas

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