FR

COMPOSITION

Acide chlorhydrique 20,00 %, iode 0,75 %, excipient hydro-alcoolique.

INDICATIONS

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UK

COMPOSITION

20.00 % hydrochloric acid, 0.75 % iodine, hydroalcoholic excipient.

INDICATIONS

Softening and partial solubilization of calculus before mechanical or ultrasonic scaling.

PROPERTIES

The use of an abrasive paste with a purely mechanical action is indicated in routine cases. It is not the same when it comes to **scaling mobile teeth**, where dental calculus is firmly attached to the tooth surface. Scaling is facilitated by the use of DETARTROL ULTRA. Due to its solvent properties, it softens calcified calculus. DETARTROL ULTRA has the following advantages:

- it deconstructs deposits of calculus which facilitates their removal.
- it **temporarily** colors organic matter, which allows the practitioner to control that complete elimination of calculus has been achieved.

CONTRAINDICATIONS

Allergic reaction to iodine.
Pregnant or breast-feeding women.

DIRECTIONS FOR USE

Carefully apply a small amount of DETARTROL ULTRA onto the surfaces to scale, using a cotton pellet.

Leave in contact for about 30 seconds.
Rinse with a water syringe.

WARNINGS AND PRECAUTIONS FOR USE

Avoid contact with mucous membranes.
When scaling is particularly difficult, especially in the presence of calculus on mobile teeth, the product can be reapplied two or three times. It is recommended to protect the gingiva and adjacent teeth.

F - Highly flammable.
Xi - Irritating to eyes, respiratory system and skin.
Keep away from sources of ignition — No smoking.
In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
Wear suitable gloves.

PRESENTATIONS

13-ml bottle
45-ml bottle

For professional dental use only

Distributor: SEPTODONT Ltd.

Unit R & S - Orchard Business Centre
St Barnabas Close - ALLINGTON, Maidstone, KENT ME16 0JZ - UK

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