FR

**COMPOSITION**

ENDOSOLV E	
Tetrachloréthylène .....	92,30 g
Excipients q.s.p. ....	100,00 g
ENDOSOLV R	
Formamide .....	66,50 g
Alcool phényléthylque .....	33,50 g

**PROPRIÉTÉS**

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UK

**COMPOSITION**

ENDOSOLV E	
Tetrachloroethylene .....	92.30 g
Excipients qs .....	100.00 g
ENDOSOLV R	
Formamide .....	66.50 g
Phenylethyl alcohol .....	33.50 g

**PROPERTIES**

Due to their respective composition, these solvents can soften traditional zinc oxide eugenol cements (ENDOSOLV E) or phenolic resin type sealers (ENDOSOLV R), when root-canal retreatment is required.

**INSTRUCTIONS FOR USE**

The operative field should be completely isolated: the use of a rubber dam around the tooth is recommended, as in all root-canal therapy.

Mechanically clean out the pulp chamber and the entrance to the canals.

Place a drop of ENDOSOLV into the chamber and clear the canal, using first a probe, then a broach, always dipping the point of the instrument in ENDOSOLV before inserting it in the canal. The instrument must be removed from the canal and wiped as soon as resistance is felt. Repeat the operation several times, until reaching the apex.

Intermediate irrigation with sodium hypochlorite is effective in neutralizing the medium.

**SPECIAL WARNINGS**

After using ENDOSOLV to remove the old filling material, it is essential to prepare the tooth again and to carefully dry it, particularly when using a zinc-oxide eugenol cement as the new filling material.

**ENDOSOLV E:**

Xn - Harmful, contains tetrachloroethylene.

N - Dangerous for the environment.

Toxic to aquatic organisms, may cause long-term adverse effects in the aquatic environment.

Limited evidence of a carcinogenic effect.

**ENDOSOLV R:**

T - Toxic, contains formamide.

Irritating to eyes.

Possible risk of harm to the unborn child.

**PRECAUTIONS FOR USE****ENDOSOLV E:**

Wear suitable protective clothing and gloves.

Do not breathe vapour.

Avoid release to the environment. Refer to special instructions/safety data sheets.

This material and its container must be disposed of as hazardous waste.

**ENDOSOLV R:**

Avoid exposure, obtain special instructions before use.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

In case of insufficient ventilation, wear suitable respiratory equipment.

This material and its container must be disposed of as hazardous waste.

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