FR

## COMPOSITION

Chlorure d'aluminium ..... 0,50 g  
 Cétrimide ..... 0,01 g  
 Excipient q.s.p. .... 100,00 ml

## PROPRIÉTÉS

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## PROPERTIES

In order to obtain impressions of high quality, it is required to operate in a clean mouth cavity thus having any traces of deposits and mucus removed from the teeth and gums.

PRE-EMP is a ready for use mouthwash solution with a pleasant taste which :

- removes effectively all the deposits and mucus adhering to the teeth, the gums and the restorations already in place,
- slightly contracts the gingival mucosa,
- momentarily stops seepage and small haemorrhages,
- disinfects and sweetens the mouth cavity.

Although PRE-EMP is harmless, it is not to be swallowed. For this reason, it should not be used in young children.

## INDICATION

Mouthwash before impression taking.

## METHOD OF USE

Ask the patient to rinse his mouth with about one tablespoonfull of undiluted PRE-EMP instructing him to whirl the liquid around the teeth and then to spit it out.

Then, ask the patient to rinse again his mouth in the same manner with a 1/3 of a glass of tepid or cold water and spit out this water. This second rinse eliminates all the oral mucus as well as PRE-EMP in excess.

The impression taking may be performed whatever the type of material used : alginates, silicones, hydrocolloids, synthetic elastomers, eugenol and zinc based pastes, plasters etc.

## IMPORTANT REMARK

A light cloud may appear in the bottle : just shake it.

## PRESENTATION

Bottle of 500 ml.

*For professional dental use only*

**Distributor:** SEPTODONT Ltd.

Units R & S – Orchard Business Center  
 St Barnabas Close Allington, Maidstone  
 Kent ME16 OJZ - UK

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