F

COMPOSITION

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Tetrafluoroethane 96.6 %
Dimethylether 3.4 %

INDICATIONS

Pulp sensitivity test.
Cooling of mixing slabs.

INSTRUCTIONS FOR USE

- Hold a cotton roll in your hand or grasp a cotton pellet with a pair of tweezers. Spray some PULPOFLUORANE onto one end of the cotton roll or onto the whole surface of the cotton pellet until they are thoroughly impregnated. Apply the refrigerated surface of the cotton roll or pellet onto the tooth to be tested, exerting pressure for 5 seconds, and continue with several short applications until a slight sensation of pain is felt.

- The cooling of the mixing slabs is obtained simply by spraying them a few seconds with PULPOFLUORANE.

SPECIAL WARNINGS

PULPOFLUORANE must never be sprayed directly onto vital teeth, nor onto the mucosa to produce cryoanaesthesia. Such an application would cause irreversible tissue reactions.

PRECAUTIONS

Do not use after the expiry date.

Pressurised container: protect from sunlight and do not expose to temperatures exceeding 50 °C. Do not pierce or burn, even after use. Do not spray on a naked flame or any incandescent material. Keep away from sources of ignition - No smoking.

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GIOVANNI OGNA & FIGLI SpA