F

COMPOSITION

Propolis, phospholipides, PEG, arôme caramel, saccharine, cétrimide.

PROPRIÉTÉS

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Propolis, phospholipids, PEG, caramel flavour, saccharin, cetrimide.

PROPERTIES

Propolis is a natural, sticky, resinous substance, which is collected by bees from various vegetable substances and processed by them to protect the hive. Bees use propolis inside the hive to seal cracks or keep it free from microorganisms, and to coat the surfaces of the honeycomb cells in order to protect the larvae against infection.

The adhesive properties of propolis, also called "bee glue", allow the gel to adhere to the mucous membranes, which provides mechanical protection.

INDICATIONS

Protection of the mucous membranes after dental treatment or in the presence of gingivitis, periodontitis, canker sores, and irritation caused by dental braces.

CONTRAINDICATIONS

Allergy to any constituent, to bee products (honey, royal jelly) and to Balsam of Peru.
Children under age 6.

INSTRUCTIONS FOR USE

Syringe: Using the applicator tip, apply a dab of gel, about the size of a pea, to the area to be treated, massaging gently. Do not rinse.

Tube: Squeeze the desired quantity of gel out of the tube, onto a mixing pad. Apply it to the mucosa, massaging gently. Do not rinse.

If required, reapply gel at a later visit.

WARNINGS AND PRECAUTIONS FOR USE

For local use only.
This product should not be used in allergic patients (see Contraindications).
This product should not be used on infected tissues unless

a suitable treatment was previously provided.
For mechanical reasons, this product should not be used if bleeding is profuse.

STORAGE

Do not store above 25°C.

PRESENTATIONS

- 1.2 g syringe with single use applicator tips.
- 6.5 g tube.

For professional use only in the practice of dentistry.

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© 05 98 260 00 01