(FR)

**UTILISATION**

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(EN)

**INTENDED USE**

GUTTASOLV is a solvent for gutta-percha.

**COMPOSITION**

Eucalyptol.

**INDICATIONS**

- Fitting of the primary gutta-percha point in the root-canal, using the lateral condensation method.
- Retreatment after root canal filling with gutta-percha.

GUTTASOLV is indicated for mature and permanent teeth.

**PERFORMANCES**

GUTTASOLV dissolves or softens gutta-percha used in endodontic treatments.

**CONTRA-INDICATIONS**

Hypersensitivity to Eucalyptol.

**INSTRUCTIONS FOR USE**

The operative field shall be completely isolated: the use of a rubber dam around the tooth is recommended.

FITTING OF THE PRIMARY POINT (LATERAL CONDENSATION):

- Select the primary gutta-percha point,
- Dip the apical tip into the solvent,
- Insert the tip into the canal and proceed as usual.

RETREATMENT:

- Place two drops of solvent into the pulp chamber,
- Remove the softened gutta-percha using a manual root canal instrument,
- Add some fresh solvent from time to time and rinse alternately with sodium hypochlorite.

**ADVERSE EFFECTS**

May cause an allergic reaction.

**WARNINGS AND PRECAUTIONS OF USE**

Close the cap securely after each use.

Warning

Flammable liquid and vapour.

Do not allow the product to enter sewers/ surface water or ground water. Refer to safety data sheet - available upon request.

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