Unimatrix Pliers

The Unimatrix Pliers is an instrument developed for an accurate and safe handling of Unimatrix and Unimatrix R rings, as well as Elastic Wedges.

Indications: The Unimatrix Pliers is used in dentistry, pediatric dentistry and integrated clinical dentistry. It has the purpose of installing Unimatrix and Unimatrix R rings, and Elastic Wedges.

Use: Unimatrix Rings: Place the Unimatrix or Unimatrix R ring in the jaws of the Unimatrix Pliers. Compress the pliers until the desired ring opening is obtained. Place the ring in the desired position and slowly open the Unimatrix Pliers. Remove the pliers from the mouth.
Elastic Wedges: Select the wedge suitable for the procedure and fit it in the pliers jaws. Compress the pliers until the required thickness of the wedge is obtained. Place the wedge in the desired position and slowly open the Unimatrix Pliers. Remove the pliers from the mouth. When finished, pull the wedge by one of its edges and cut it in the middle, removing resulting parts from the patient's mouth.

Warnings, precautions and restrictions: Professional use only. When there is proven allergy to any of its components, the product must not be used. For a longer useful life of the instrument, it is recommended to sterilize it by physical means. The utilization of the pliers for other purposes is the full responsibility of the user. The pliers must be sterilized before its first use and after each patient. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Cleaning and sterilization: Sterilizable by autoclave (126°C - 16min / 134°C - 18min). Before sterilization, proceed with the cleaning using water and enzymatic detergent. Cleaning: Carry out the cleaning with the aid of water and enzymatic detergent 4 enzymes (Amylase, Protease, Lipase and Carbohydrase) and with pure pH range between 6.5 - 7.5. Prepare the solution according to the manufacturer's instructions. The water should be potable. For a better effectiveness, it is recommended that the temperature be between 40°C and 55°C. Submerge the products and leave in contact for 5 minutes with the solution. Remove parts and rinse under running water for at least 10 seconds. Dry the pieces with absorbent paper or disposable wipes. Inspection: Check the product before each use, observing whether there are cracks or breakage, and do not use in these cases. These conditions are indicative of fragility of the material, whose strength can be reduced during clinical use.

Storage: To be stored in dry and fresh place, protected from sunlight.

Composition: Stainless steel.

Applied Standards: The Unimatrix Pliers are resistant to corrosion, as ISO 13402 Standard.

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Obelis S.A.