# Restofill Bulk Flow <sup>fr</sup>

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# Restofill Bulk Flow <sup>en</sup>

Light-cured flowable  
bulk fill restorative material

## DESCRIPTION

Restofill Bulk Flow is a visible-light activated, radiopaque, flowable bulk fill restorative material. It can be placed in 4mm increments.

## COMPOSITION

- Syringe, 2 gr.
- Bis-EMA / UDMA / Barium glass / Silica
  - The percentage by weight of total inorganic filler (particle size 0.01~2.5µm) is ca. 65%

## INDICATIONS

- Base/Liner under direct restorations (Class I, II)
- Blocking out of undercuts
- Repair of composite/ceramic veneers
- Anterior restorations (Class III, IV)
- Class V restorations (cervical caries, root erosion, wedge shaped defects)

## CONTRA-INDICATIONS

Restofill Bulk Flow should not be used with patients who develop hypersensitivity to it or have a history of hypersensitivity to methacrylate monomer.

## RECOMMENDED PROCEDURE

### 1. Shade selection

Clean the teeth with pumice and water to remove surface stains or extraneous plaque. Teeth are not monochromatic. Consider shade and restoration depth. It is desirable to choose the shade after mocking up. Alternatively, Vita Lumin® Vacuum shade guide may be used. Using of rubber dam is recommended for isolation.

### 2. Cavity preparation

- **Anterior restoration**  
Using the conventional acid etching, prepare cavity for all Class III, Class IV, and Class V restoration.
- **Posterior restoration**  
Prepare the cavity. No residual amalgam or other base material should be left on the internal surfaces of preparation that would interfere with light transmission and the hardening of the restorative material.

### 3. Pulp protection

In deep cavities cover the dentin close to the pulp with a minimum amount of calcium hydroxide liner leaving the rest of cavity surface free bonding. Glass ionomer or other eugenol-free base materials may be used, if wished.

### 4. Placement of matrix

Use a matrix system, preferably a transparent one, with proper wedging for proximal contacts. Pre-wedging is advocated to achieve slight separation and facilitate optimal proximal contact.

### 5. Enamel and dentin treatment

Follow the manufacturer's instructions regarding etching, priming, adhesive application and curing.

### 6. Placement

After removing cap, insert the disposable tip onto the syringe. Before applying resin, make sure that needle of tip is not plugged. If plugged, remove the tip and express a small amount of resin directly from the luer of syringe. Remove any visible plug, if present, from the syringe. Replace new disposable tip and again express resin. Place resin up to 4mm or less into the prepared cavity. Restofill Bulk Flow may be extruded onto a dispensing container and applied with a brush or other instruments. To prevent oozing, pull back on syringe plunger after dispensing procedure.

### 7. Curing

Expose each area of restoration surface to a high intensity visible light source (wave length 420 to 500nm) for 40 seconds. Hold the light guide tip as close to the restorative as possible during light exposure. Use a light curing unit with a light intensity of 550mW/cm<sup>2</sup> or higher.

### 8. Finishing

Immediately after curing contour restoration surfaces with fine finishing bur or stone. Carefully adjust occlusion by removing material with a fine polishing bur or stone.

## STORAGE

1. Do not store at elevated temperature or intense light.
2. Keep in a cool place.
3. Do not store materials in proximity to eugenol containing material.
4. After using Restofill Bulk Flow, it should be capped immediately.

## PRECAUTIONS

1. For dental use only.
2. It should be used by a dentist or dental professional. Observe the procedure and the precautions.
3. Keep it away from a reach of non-dental users including children, senior citizens and others.
4. If skin sensitization occurs allergy to acrylate, stop using resin.
5. Exposed to uncured resin should be avoided. If resin contacts the skin, wash immediately with soap and water.
6. If accidental contact with eyes or prolonged contact with oral tissues occurs, flush immediately with a large amount of water.
7. If refrigerated, the material must be allowed to reach room temperature before use.
8. Use of protective gloves and a no-touch technique is recommended.
9. Protective device for the respiration tract is highly recommended before using.
10. In case of non-polymerization, check the light intensity of the light curing unit and then contact the nearest dealer(s) or manufacturer directly regarding the same quality matters.
11. Do not use after expiry date.
12. Restofill Bulk Flow should be used within 5 minutes as soon as the material is extruded from the syringe.

## WARRANTY

Manufacturer will replace product that is proven to be defective. Manufacturer does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

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VERICOM CO., LTD.