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**COMPOSITION**
Acetone, Ethyl acetate, Phenylethyl alcohol, Amyl acetate.

**INDICATIONS**
HYDROL is a dehydrating and degreasing agent for:
- dental cavities before filling
- dental preparation.

**PERFORMANCES**
HYDROL simultaneously removes moisture and all traces of grease.
HYDROL is economical in use and permanently solves one of the practitioner's constant problems.

**CONTRA INDICATIONS**
Hypersensitivity to any component of the formulation.
Use on vital teeth.

**ADVERSE EVENTS**
Allergic reaction.

**INSTRUCTIONS FOR USE**
The operative field should be completely isolated: the use of a rubber dam around the tooth is recommended.
Use the adapted container for the sampling.

Dab the area to be dried and degreased with a cotton ball soaked in HYDROL.

Wait a few seconds for complete evaporation of the liquid.

**WARNINGS AND PRECAUTIONS FOR USE**
Danger
Contains ethyl acetate and acetone.
Highly flammable liquid and vapour.
Causes serious eye irritation.
May cause drowsiness or dizziness.
Refer to safety data sheet (available upon request).

**IMPORTANT REMARKS**
Before filling the tooth with a temporary self-curing paste or a permanent acrylic resin filling material, wait until the cavity is completely dry.
Do not apply excessive amount of HYDROL.
HYDROL is not designed for degreasing the gingival mucosa, as prolonged application may cause a burning sensation.
Carefully close the bottle after use.

**PRESENTATION**
Bottle containing 45 mL of solution
Bottle containing 250 mL of solution

For information:
The content of the bottle containing 45 mL of solution is estimated to ensure an average of 1956 uses.
The content of the bottle containing 250 mL of solution is estimated to ensure an average of 10869 uses.

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